Single-use Blades
DYONICS® Straight BladesDYONICS Straight Blades
Packaged 6 per box, sterile

Power/EP-1® REF	Description	Latch Color
7205305	3.5 mm Full Radius	Beige
7205306	4.5 mm Full Radius	Yellow
7205307	5.5 mm Full Radius	Orange
7210751	3.5 mm Full Radius ELITE	Beige
7210499	4.5 mm Full Radius ELITE	Maroon
7210909	5.5 mm Full Radius ELITE	Olive
7205315	4.5 mm TURBOTRIMMER	Powder blue
7205345	4.5 mm INCISOR® Plus	Violet
7205459	5.5 mm INCISOR Plus	Teal
72200095	3.5 mm INCISOR Plus ELITE	White
7210976	4.5 mm INCISOR Plus ELITE	Slate
72200081	5.5 mm INCISOR Plus ELITE	Lilac
7209211	3.2 Chondroplasty Blade	

Kopija tikra

2015-01-02

Vadgiminkas
Klaudas Ročys

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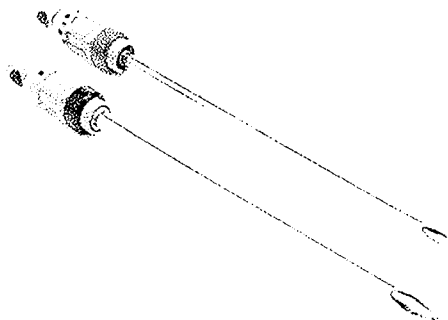
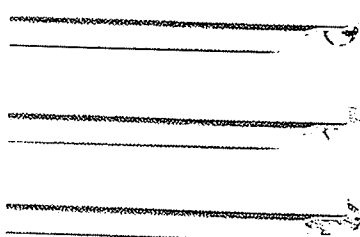
Kopija tikra

Hip Shaver Blades and Burrs

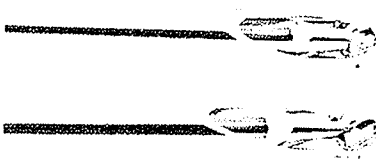
DYONICS® High Visibility and Extended Length Blades and Burrs

DYONICS HIGH VISIBILITY Hip Burrs
Packaged 3 per box, sterile

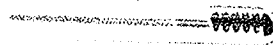
Reference #	Description	Latch Color
72203127*	5.5 mm HIGH VISIBILITY Sheath Abrader Burr	Black
72203128	4.0 mm HIGH VISIBILITY Sheath Abrader Burr	Purple
72203129*	5.5 mm HIGH VISIBILITY Sheath Flat Top Burr	Black
72203130	4.0 mm HIGH VISIBILITY Sheath Flat Top Burr	Purple
72203131*	5.5 mm HIGH VISIBILITY Sheath Barrel Burr	Black
72203132	4.0 mm HIGH VISIBILITY Sheath Barrel Burr	Purple

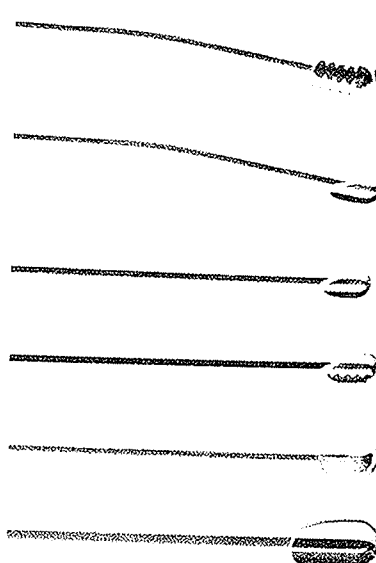
DYONICS POWER Shaver Burrs, 18 cm working length, partial hood.
Packaged 3 per box, sterile

Reference #	Description
72200080*	4.0 mm Long Abrader
72200082*	5.5 mm Long Abrader

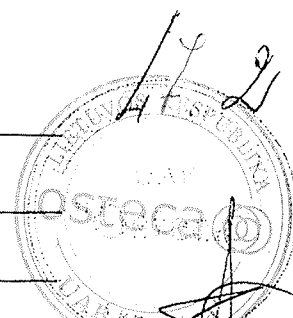
DYONICS POWER Shaver Blades, 18 cm working length
Packaged 3 per box, sterile

Reference #	Description
72200414*	4.5 mm Long INCISOR® Plus ELITE
72200494*	4.5 mm Long Curved INCISOR Plus ELITE
7205335*	4.5 mm Long Curved Full Radius Concave
7205336	4.5 mm Long Curved Full Radius Convex
7205341*	4.5 mm Long Full Radius
7205343*	4.5 mm Long INCISOR
7210979*	4.5 mm Long BONECUTTER®
6900678*	5.5 mm Long Full Radius

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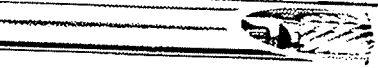
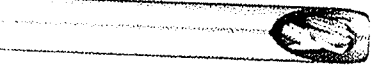
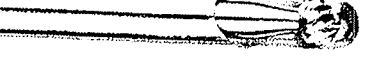
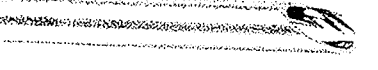
* Item shown

† Requires the DYONICS POWERMAX ELITE Handpiece. Reference #s 72200616, 72200617 and 72200872 on page G-2.

Call +1 800 343 5717 (U.S.) or contact your local representative

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Ref.	Apibūdinimas	spalva	APM maksimumas
7205324	4,0 mm Abrader	vandens	8,000
7205325	5,5 mm Abrader	juoda	8,000
72200722	4,0 mm ELITE Abrader	vandens	10,000
72200723	5,5 mm ELITE Abrader	juoda	10,000
7207906	4,0 mm Abrader, kairiarankiams	bespalvis	8,000
7205326	4,0 mm Acromionizer	rausvai violetinė	8,000
7205327	5,5 mm Acromionizer	ruda	8,000
7209512	4,0 mm Acromionizer	rausvai violetinė	8,000
72200724	4,0 mm ELITE Acromionizer	rausvai violetinė	10,000
72200725	5,5 mm ELITE Acromionizer	ruda	10,000
7205330	4,0 mm STONECUTTER	kaštoninė	8,000
7205331	5,5 mm STONECUTTER	gelsvai žalia	8,000
72200728	4,0 mm ELITE STONECUTTER	kaštoninė	10,000
72200729	5,5 mm ELITE STONECUTTER	gelsvai žalia	10,000
7209069	5,5 mm STONECUTTER, kairiarankiams	bespalvis	8,000
7205328	4,0 mm NOTCHBLASTER	alyvinė	8,000
7205329	5,5 mm NOTCHBLASTER	rausvai oranžinė	8,000
7220726	4,0 mm ELITE NOTCHBLASTER	alyvinė	10,000
72200727	5,5 mm ELITE NOTCHBLASTER	rausvai oranžinė	10,000
7205668	4,0 mm ACROMIOBLASTER	pilkai žalia	8,000
7205669	5,5 mm ACROMIOBLASTER	plytinio raudonumo	8,000
72200730	4,0 mm ELITE ACROMIOBLASTER	pilkai žalia	10,000
72200731	5,5 mm ELITE ACROMIOBLASTER	plytinio raudonumo	8,000
7209335	4,5 mm Shielded	balta	8,000
6900988	4,0 mm kūginis	bespalvis	8,000
6900996	5,5 mm kūginis	bespalvis	8,000
7205727	4,5 mm HELICUT	pilkai melsva	5,000
7210460	4,5 mm HELICUT, kairiarankiams	bespalvis	5,000

47.3

2015-01-02

Veddybūnukas
moldas Roči

Kopija tikra

CHART REFERENCES

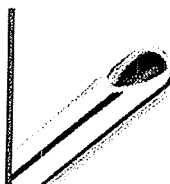
STANDARD

Full radius: ideal for soft tissue resection and debridement.

3,5	FR1035ST
4,2	FR1042ST
4,8	FR1048ST
5,5	FR1055ST

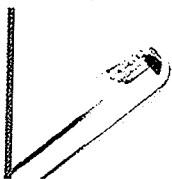
SAGA

Gentle soft tissue resection. The protective window gives a better accuracy in preservation of nearby structures like cartilage.

4,5	FR1045SG
-----	----------

LEGEND

Aggressive meniscal resection, fast removal of soft tissue for shoulder. Indicated for combination of soft tissue and bone resection.

3,5	FR1035LG
4,2	FR1042LG
4,8	FR1048LG
5,5	FR1055LG

EXPRESS

Oval bur. Fast bone resection. Indicated for subacromial decompression.

4,0	FR1040EX
6,0	FR1060EX

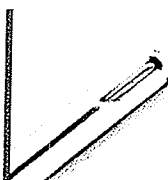
ICON

Spherical bur. Ideal for notchplasty and osteochondral debridement.

4,5	FR1045IC
5,5	FR1055IC

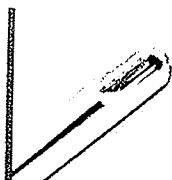
DIVA

Wide full radius for extensive soft tissue resection.

4,5	FR1045DV
-----	----------

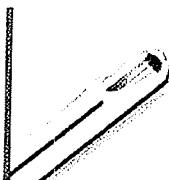
EPIC

Indicated for resistant tissues in knee.

4,5	FR1045EP
-----	----------

SYMBOL

Intermediate full radius, efficient in all soft tissues procedures.

4,5	FR1045SM
-----	----------

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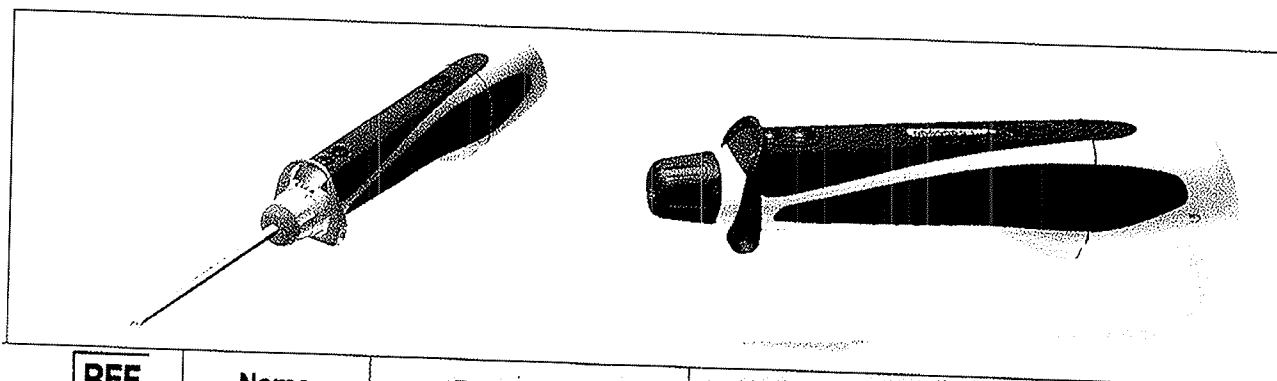
2015-01-02

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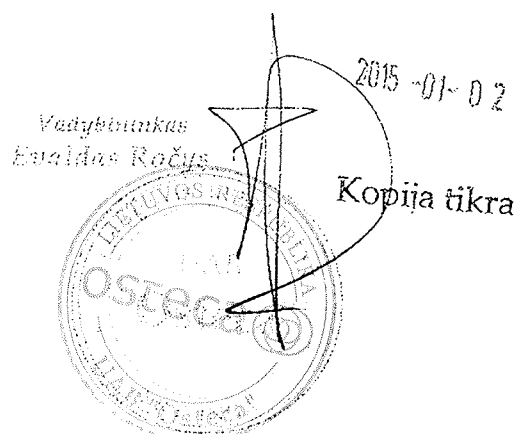
Kopija tikra

REF	Name	Description	Function	Supplied as
FR20700	REMOVAL BOX	This is a plastic-coated box supplied inside the sterile package of the TOOL BODY.	The REMOVAL BOX allows the user to remove the central unit from the TOOL BODY after the surgical procedure.	Sterile SINGLE-USE
REF				
	Name	Description	Function	Supply characteristi
FR20800	INSERTER BOX	This is a plastic-coated box supplied inside the sterile package of the TOOL BODY.	The INSERTER BOX allows the user to insert the central unit into the TOOL BODY before the surgical procedure.	Sterile SINGLE-USE

The system is delivered inside a package containing:

- 1 CHARGING BASE complete with power supply cable;
- 1 INTERNAL CENTRAL UNIT;
- 1 TOOL BODY  **SINGLE-USE Component (STERILE package)**

Kopija tikra

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FR20700 Vienkartinio naudojimo šeiverio rankena, STERILI

4.4.4

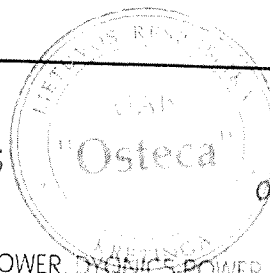
Kopija tikra

2015-01-02

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DYONICS® POWER Shaver Systems

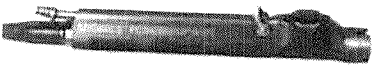
DYONICS Shaver Handpieces

2015-03-23

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For use with DYONICS POWER, DYONICS POWER II and EP-1 Control Units

Reference #	Description
 72200616*	DYONICS POWERMAX® ELITE Handpiece with blade multi-positioning, hand controls, suction lever, 10 ft (3 m) cord
 72200617*	DYONICS POWERMAX ELITE Handpiece with blade multi-positioning, no hand controls, suction lever, 10 ft (3 m) cord
 72200872	DYONICS POWERMAX ELITE Handpiece hand-controlled with opposite-side suction lever and stiffer buttons
 72201500†	POWERMINI® Small Joint Handpiece with blade multi-positioning, hand controls, suction lever, 10 ft. (3m) cord For use with DYONICS POWER II Control System.
 72201503†	POWERMINI Small Joint Handpiece with blade multi-positioning, no hand controls, suction lever, 10 ft. (3m) cord For use with DYONICS POWER II Control System.
 7205357	DYONICS Mini-motor Handpiece, suction lever, 10 ft (3 m) cord
 7205971‡	DYONICS POWER Ultralight Handpiece, suction lever, 10 ft (3 m) cord

445

* Handpieces accept all blades

† Handpieces only work with the Single-use Extremity Blades and Burrs listed on page D-11 and F-11.

‡ Handpiece accepts all straight blades, but only ULTRA series curved blades.

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⚠ WARNINGS

- Smith & Nephew DYONICS® Disposable Arthroscopic Surgery Blades are packaged as a set. They must be used as supplied. Do not interchange blade components.
- Irreversible damage will result from any attempt to disassemble curved blades.
- Irreversible damage to burrs will result if they are run without the flow of irrigation (dry).

Blade Selection

All Smith & Nephew DYONICS Disposable Arthroscopic Surgery Blades have been extensively tested for safety and performance. Based on these tests, shaver blade speed ranges have been preset in the control unit (Table 1).

Tissue cutting performance has been determined at various speeds for each blade. This information indicates that Oscillate mode is more efficient for cutting soft tissue (Figure 10), while Forward mode is more efficient for cutting hard, osseous tissue (Figure 11).

Note: All blade speeds can be set in increments of 100 rpm in Forward/Reverse mode.

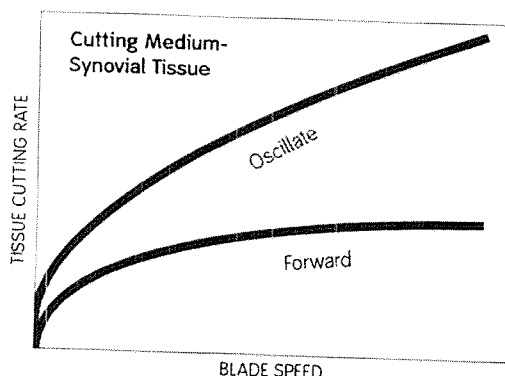
Figure 10 Cutting Efficiency with Soft Tissue

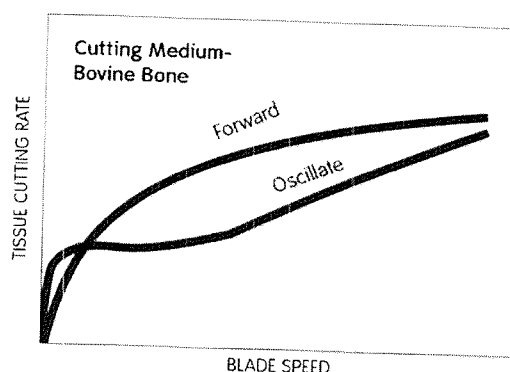
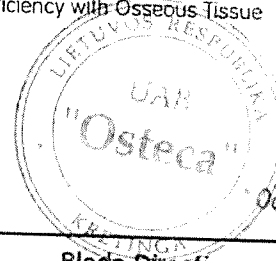
Figure 11 Cutting Efficiency with Osseous Tissue

Table 1. Programmed Blade Speeds

Blade Type	Default Set Speed	v3.0 Minimum	Maximum	Blade Direction
Straight Blades	2000	100 <i>47.5</i>	5000*	Forward/Reverse Oscillate
	2000	500	3000 <i>47.5</i>	
Curved Blades	2000	100	5000	Forward/Reverse Oscillate
	2000	500	3000	
Ultra Series Curved Blades	2000	100	5000	Forward/Reverse Oscillate
	2000	500	3000	
Burrs and Series 3000 Blades	4000	500	8000†	Forward/Reverse Oscillate
	2000	500	3000	
ELITE® Burrs	5000	500 <i>47.5</i>	10,000	Forward/Reverse Oscillate
	2000	500	3,000	
Mini Blades	2000	100	3500	Forward/Reverse Oscillate
	1000	500	3000	

* Excluding Smith & Nephew TURBOWISKER® Blades (REF 7205316) which run to a maximum of 3000 rpm.

† Excluding Smith & Nephew HELICUT® Blades (REF 7205727), which run to a maximum of 5000 rpm.

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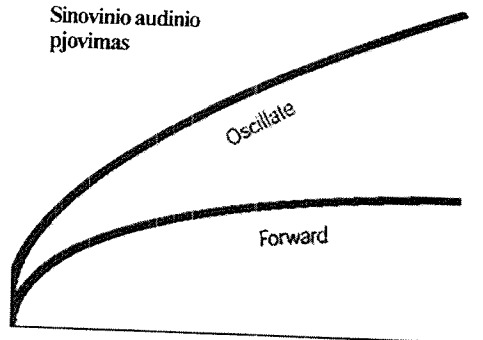
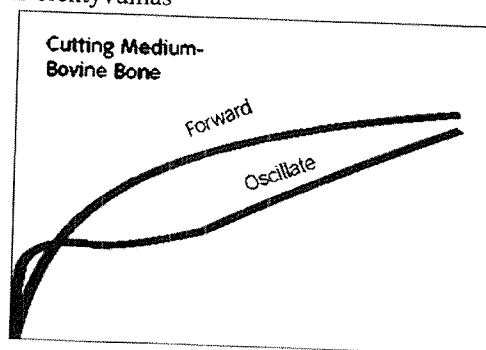
Glossary of Symbols

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician

Equipment Classification—
Type B Equipment

Non-Sterile

Caution: See Instructions for Use

Suction On
Fluid flow through
handpiece unrestricted

Catalog Number
Part number or reorder
number

Dangerous Voltage:
Danger of electric shock

Suction Off
Fluid flow through
handpiece blocked

Date of Manufacture

Indicates explosion risk in
the presence of flammable
anesthetics

Diagnostic Display Button
Displays diagnostic codes

Serial Number

Power On Connection to
the mains

Forward
Clockwise direction of
shaver rotation (distal view)

UL Classification

Power Off Disconnection
from the mains

Reverse
Counterclockwise direction
of shaver rotation (distal
view)

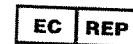
CE mark

Slower
Decrease shaver speed

Oscillate
Alternating
forward/reverse direction
of shaver rotation

European Representative

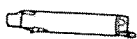
Faster
Increase shaver speed

Blade Window Lock
Adjustment of cutting
window stop position

This Way Up

Handpiece
Shaver motor drive unit

Non-ionizing
electromagnetic radiation

Fragile, handle with care

Footswitch
Foot pedal control

Fuse

Keep Dry

Minimum
Lowest shaver speed value

Equipotentiality

Temperature Range

Maximum
Highest shaver speed
value

Watertight equipment per
IEC 529

Humidity

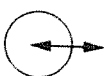
Speed
Shaver rotation rate in
revolutions per minute

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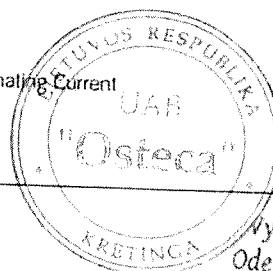
UAB "Osteca" vadovė
D. Raklevičienė

Forward/Reverse Speed Shift

Depressing and holding the Forward or Reverse buttons on the handpiece for one second will alternately shift between two pre-set speeds (high and low) in the forward or reverse direction. High and low speeds are set by pushing the Slower or Faster buttons on the front panel of the Smith & Nephew DYONICS® POWER System Control Unit in either the slow or fast speed. If the slow and fast speeds are not set, then depressing and holding the handpiece buttons will not activate Speed Shift.

Note: Speed Shift is not applicable in Oscillate mode and does not work through any footswitch.

Footswitch

The footswitch is required when using the Standard Shaver handpiece, the UltraLight Shaver Handpiece, or the Mini Shaver Handpiece, and can be optionally used with the Hand Control Shaver Handpieces.

The shaver system uses a standard low profile footswitch (REF 7205399) to control blade action (Figure 6). The footswitch, which plugs into the connection port on the left side of the control unit front panel, has three circular buttons (REVERSE, OSCILLATE, and FORWARD) and a smaller button for blade WINDOW LOCK.

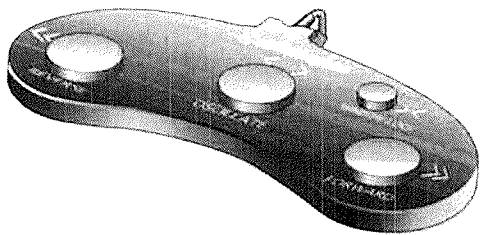
Figure 6 Smith & Nephew DYONICS® POWER System Footswitch

Depressing the FORWARD or REVERSE button causes the control unit to supply power to the handpiece to drive the blade in the direction indicated. The control unit accelerates the blade rotation until it reaches the set speed and maintains that speed until you release the button. The footswitch buttons are not throttles, but simply turn the motor drive ON or OFF in a specific direction.

Note: The footswitch does not control the Forward or Reverse Shift Speed settings.

The OSCILLATE button causes the control unit to send alternate forward and reverse commands to the handpiece. The control unit reverses direction when the blade rotation reaches the set speed. The control unit continues the alternating signals until you remove your foot from the pedal.

Blade Window Lock

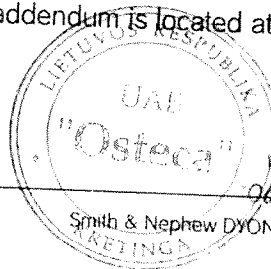
Blade Window Lock sets the stop position of the inner blade relative to the opening at the top of the outer blade. Each time you press the WINDOW LOCK button, the blade rotates clockwise to a new stop position. Blade WINDOW LOCK can also be set by simultaneously pressing the Slower and Faster buttons on the shaver control unit, or by pressing and holding the Oscillate button on the Hand Control Shaver Handpieces.

Two optional footswitch models are available for use with the shaver system:

- The Accelerator Footswitch (REF 7205397) allows the Forward, Reverse and Oscillate buttons to control speed as well as blade direction.
- The Pedal Style Footswitch (REF 7205396) features two foot pedals to control blade action.

Smith & Nephew DYONICS Powered Instruments

The Smith & Nephew DYONICS POWER Shaver System provides the convenient capability to drive powered instrumentation (i.e., drill/wire drivers, sagittal saw, etc.) as well as a selection of associated blades and drill bits when purchased separately. If these items have been purchased, please read the Smith & Nephew DYONICS Powered Instruments Addendum. The addendum is located at the end of this manual.

Kopija tikra

2015-02-09

Mr. Vadybininke

Odeta Raklauskaitė

Smith & Nephew DYONICS® POWER Shaver System Operations/Service Manual

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Introduction

The Smith & Nephew DYONICS® POWER Shaver System is a highly versatile surgical tool designed to meet the most demanding requirements of abrasion arthroplasty, synovectomy, and intra-articular cutting and shaving.

A variety of disposable straight and curved surgical blades and burrs can be inserted into the shaver handpiece for various surgical requirements. A footswitch is used to control blade action, selecting forward, reverse or oscillate. The control unit provides settings for the blade speed within minimum and maximum speeds, preprogrammed for each blade type. The optional Hand Control Shaver Handpiece allows the surgeon to control shaver operation without the need for a footswitch. The optional Mini-Shaver Handpiece accepts a special selection of blades designed for small-joint procedures, and for Functional Endoscopic Sinus Surgery (FESS).

Note: The Smith & Nephew Ultra Series curved blades and the Ultra Series TURBOWHISKER® blade may be used in all Smith & Nephew shaver handpieces. Standard curved blades and the standard TURBOWHISKER blade will not fit Smith & Nephew DYONICS UltraLight Handpieces.

The Smith & Nephew DYONICS POWER Shaver System features many performance and ease-of-use advantages, including:

- Intuitive front panel display and controls provide an immediate understanding and control of system performance.
- The system automatically detects the blade type in use and sets the operating parameters accordingly.

- When you change blades, Blade Family Recall automatically returns to the previous speed settings in Forward/Reverse and Oscillate modes (blade types are not recognized by the UltraLight Handpiece).

- Speed Shift offers the ability to alternate between user-defined, preset low and high speeds by simply depressing and holding the reverse or forward buttons on the handpiece, respectively (Hand Control Shaver Handpieces REF 7205354, REF 7209891, REF 72200616, and REF 7210542).

- The low profile footswitch enables more comfortable control of the system over a sustained period.

- Blade Window Lock, which can be set from the control unit, footswitch or hand control shaver handpieces, determines the stop position of the inner blade relative to the opening in the outer blade.

- Optional HERMES® Digital O.R. Control Center voice command and/or touch screen control.

- The optional Hand Control Shaver Handpieces (REF 7205354, REF 7209891, REF 72200616, and REF 7210542) feature push button controls that can be used instead of the footswitch controls.

- A latch button on the blade collar ejects the blade for faster, easier blade changes.

- A suction lever mounted on the handpiece, which operates from fully opened to fully closed, allows manual control over outflow.

- Front panel connectors for the footswitch and the handpiece are field replaceable.

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