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Instructions of use

X-Ray Protective Equipment

We thank you for using Medical Index equipment. Before using it, we recommend you to examine the following instructions with a strong attention:

If you desire any further information concerning the Medical Index radiation protective clothing, please do not hesitate to contact us.

Caution

The equipment are personal protective equipment against X-rays exposure caused by x-rays generating tubes used in radiology and are intended to reduce the x-ray exposure of the wearer.

As personal protective equipment, their choice and use has to conform to the european council directive no. 89/656/EEC of 30.11.89, under the authority of a responsible competent in radio protection.

Recommendations and Limitations on Use

Recommendations are given for each type of equipment in the joined table I. Wearer has to check if he is using an equipment adapted to the risks entailed. A particular attention has to be brought on the choice of the lead equivalent thickness of the equipment and on the choice of front and back protection.

Table I Personal Protective Equipment Against X-Rays

Trade Name	Model Code	Sizes available (1)	Lead equivalent thickness in mmPb/ 100kV (2)		Recommendations and limitations for use
			Front side	Back side	
Coat Apron	CA09050/25	S, M, L	0,25	0,25	For frontal protection only. To be used by operators in X-Ray examinations room during radiological examinations with or without interventional procedures, where the radiation only comes in the front Intended to protect the main part of the body of the operator. Has to be used with additional protective devices to protect the full body: <u>Thyroid Shields, protective eye glasses or helmets.</u>
	CA09050/35		0,35	0,25	
	CA09050/50		0,50	0,25	
					For protection of the front and back parts of the body.

Double Side Apron	DA090107/50	S, M, L	2x0,25 0,25 = 0,50 (3)	To be used by operators in X-Ray examinations room during radiological examinations with or without interventional procedures. Intended to protect the main part of the body of the operator. Has to be used with additional protective devices to protect the full body: Thyroid Shields, protective eye glasses or helmets.
Skirt and Vest	SV105107/50	S, M, L	2x0,25 0,25 = 0,50 (3)	For protection of the front and back parts of the body. To be used by operators in X-Ray examinations room during radiological examinations with or without interventional procedures. Intended to protect the main part of the body of the operator. Has to be used with additional protective devices to protect the full body: Thyroid Shields, protective eye glasses or helmets.
Kids Apron	KA05045/50	S, M, L	0,50 0,50	For frontal protection only To be used by children patients during Intended to protect the main part of the body and more particularly reproductive organs. Has to be used with additional protective devices to protect the full body: Thyroid Shields, protective eye glasses or helmets
Thyroid Shield	TS00405/35 TS00405/50	S, M, L	0,35 - 0,50 -	To be used in X-Ray examination room during radiological examinations to cover the throid. Should be used along with the apron. Check that an overlapping is realised between the junction of these equipment and that no part is uncovered.
Gonad Protection	GP02025/50	XS, S, M, L	0,50 -	To be used by the patients during radiological examinations, to cover the gonads

(1) The corresponding human body dimensions for each size are given in table 2

(2) For radiation quality of 100 kV with a copper filtration of 0,2 mm

(3) Protection equivalent to 0,50 mmPb by overlapping of two layers with a lead equivalent thickness of 0,25 mmPb

The indications of the present instructions for use can, on no account, substitute to the prescripts set up by the responsible competent in radio protection.

The garments are manufactured according to the standard IEC 1331-1 and IEC 61331-3 which include testing up to a voltage of 100 kV with a copper filtration of 0,2 mm. So, the garment confirms to a minimum lead equivalent thickness as stated on the sewn-in label. For this reason, we guarantee you a suitable protection for X-ray tube voltage at 100 kV with a copper filtration of 0,2 mm. If you wish to use this equipment at another voltage, please contact us.

Ergonomics

-

These equipment have to be adapted to size wearer and correctly adjusted to be effective. The following tables give you the size system:

Size designation

Aprons and Thyroid Shield

Skirt

Size	Height	Chest girth (cm)	Size	Height (cm)	Waist girth (cm)
S	145-175	70-99	S	145-175	56-80
M	145-195	100-119	M	145-195	80-100
L	155-195	120 and above	L	155-195	100 and above

Kids Apron

Size	Height	Age (Years)
S	< 100	3-5
M	100-125	6-8
L	125-145	9-12

Gonad Protection

Size	Height (cm)	Waist girth (cm)
XS	< 145	< 56
S	145-165	56-80
M	145-195	80-100
L	155-195	100 and above

Storage and Life Expectancy

The life expectancy of this equipment is ranging from 5 to 10 years depending very strong on their use and store conditions. For this reason, we recommend:

- Avoid exposure to direct heat and high relative humidity
- Hang Apparel on a hanger after use. Do not fold, crease or wrap it to avoid permanent creasing
- Do not hang on hooks or nail as this will distort the apparel and lead vinyl

For transporting , we recommend you to fold it carefully and place it in its original packaging or an equivalent one.

Cleaning

The special coating of the Medical Index radiation protective clothing prevents that dirt penetrates into the material.

To avoid a damage of the coating

- Clean regularly for dust with sponge or cloth wet in a solution of mild soap and water
- Clean wet stains with a lukewarm detergent and a soft cloth.
- Clean dry stains with a lukewarm detergent, which works in for about 60 minutes,. Then clean with a soft cloth.
- For extremely persistent stains we recommend to use a sodium thiosulfate 10% aqueous solution. Conform to the instructions for use of the product when using it.
- **Please do not use caustic cleaning agents such as alcoholic solvents, gasoline or dilution and do not soak in water or keep under running tap.**

Disinfecting

Do not use any disinfecting with alcohol and do not autoclave.

Inspection

Please inspect your garment thoroughly upon receipt and before each use. We advise you to have a particular attention to :

- Visual inspection: we advise you to:

check the lead equivalent and size labels sewn in the garment to make sure you are using the correct garment.

Check equipment for any visible defects in workmanship and notify us immediately if your garment is not satisfactory. Light spots, small or large, may appear which indicate localised thicker particles. This effect increases x-ray absorption and should be of no concern. A small amount of dark spots/pin holding can appear which is inherent to the manufacturing process and as long there are no dark spots exceeding 5 mm in diameter health physicist have determined they are of no concern. This should not be interpreted as defective manufacturing. **There should be concern if you find dark spots or holes larger than 5 mm or an area which shows many small spots clumped together as well as if dark lines appear as if they usually indicate possible cracks thinning in the material.**

Please contact Medical Index directly in case of inadequacy.

- Radiological Inspection: we advise you to verify regularly the conservation of the radioprotection properties (homogeneity or the attenuation).

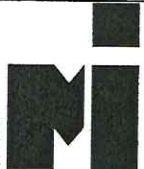
Copy of Label

A label is sewn on each equipment and located:

- 10 cm underneath the left of the sleeve-hole for aprons
- 10 cm underneath the waist for the model Skirt
- at the bottom side for the Gonad Protection and the Thyroid Shield

You will find the following information on it:

- (1) Manufacturer - Responsible for placing product on the market
- (2) Model
- (3) Size: length x width in cm, S, M, L for small, medium or large
- (4) Reference of the Model
- (5) Serial Number (two digits for the months/two digits for the year)
- (6) Lead equivalent thickness for the front side of the garment under 100 kV with copper filtration of 0,2 mm
- (7) Lead equivalent thickness for the back side of the garment under 100 kV with copper filtration of 0,2 mm
- (8) Number of the specific standards
- (9) CE marking and code of organisation in charge of monitoring:
TÜV Product Service GmbH
Ridlerstraße 31
80339 München
- (10) Pictogram meaning that the customer has to look at the instruction for use and more particularly at the cleaning instructions.
- (11) Pictogram for the size


medical index GM BH
 ...made for friends
 Germany

Model:.....
Size:.....
Reference No.:
Serial No.:.....
Front Side:.....
Back Side:.....

According to IEC 1331-1 and IEC 61331-3

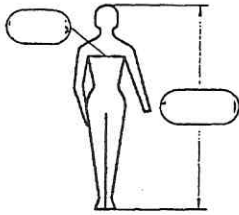 0123

Copy of labels

Label is located 10 cm underneath of the left the sleeve-hole, 10 cm underneath of the waist (Skirt) or at the bottom side of the Gonad Protection.

"MATRIX" for X-Ray Protection Garments

Finally the solution for strong load with disinfection agents

The MATRIX tissue was successful tested with the following disinfection agents:

**Betadine
(Betaisodona)
Octenisept
Octeniderm
Kodan Tinktur forte
Betaseptic Mundipharma
Sterillium classic pure
Desderman pure
u.v.m.**

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„TUV Suddeutschland Bau und Betrieb GmbH“